

ZG Lift Assist Recliner Instructions and Setup

L A **Z** B O Y[®]
H E A L T H C A R E



Initialization

Before using the chair, please initialize the system by pressing and holding the Zero Gravity and Sleep buttons until the system stops beeping and chair stops moving.

Disconnections

If your cords should ever become disconnected from the control box or wall, reset your system by initializing again.

Troubleshooting


Please check the control box to ensure it has power. When plugged into a mains power (wall), the LED indicator will be green. If the power source is battery only, the LED will be off.



If all cords are properly seated and power is connected to the chair, please initialize it once again.

Please check to ensure all cords are plugged in and fully seated. No colored **o-rings*** should be visible.

Properly seated motor cables

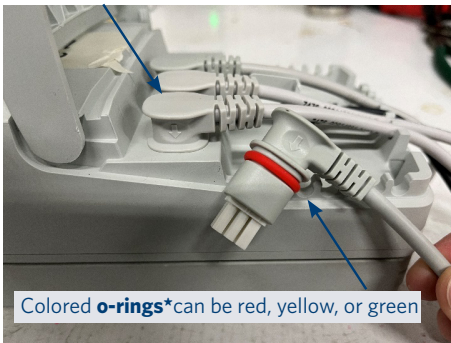

Colored **o-rings*** can be red, yellow, or green

Properly seated power cord



Disconnected power cord



Battery Information

The battery will signal its last 10% of charge by emitting a tone when any function of the chair is operated.



LED	Indication of Operation
Solid yellow	Charging
No LED light	Fully charged
Flashing yellow	Error during charging